

Portable X-Ray Zero Footprint Viewer User Guide

Logging In

1. Open your web browser and go to <https://images.portablexray.co.nz>
2. Enter your username and password.
3. Click Enter to sign in.

Voyager ZeroFP Login



Username:

Password:

For Clinical Review Only.

[Register](#)



After logging in, the main worklist will open.

Time:

Patient Name:

Radiology UR:




voyager

Patient Name	DOB	Study Date	Study Name	Type	Referred By	UR
No matching studies.						

Forgotten Password


Email pacs@portablexray.co.nz and a temporary password will be sent to you.

Searching for a Patient

Search using either of the following:

- Patient name (LAST NAME,FIRST NAME) capitals, no spaces
- Radiology UR/NHI

Enter your search term and press Enter.

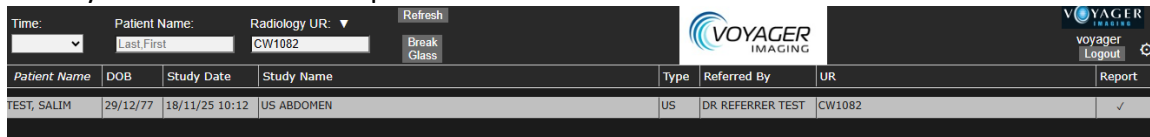

The screenshot shows the top section of the Voyager Imaging interface. It includes a search bar with fields for 'Time', 'Patient Name' (with a 'Last, First' dropdown), and 'Radiology UR' (with a dropdown). There are 'Refresh', 'Break Glass', and 'voyager Logout' buttons. Below the search bar is a table with columns: Patient Name, DOB, Study Date, Study Name, Type, Referred By, UR, and Report. A single row is visible with the following data: TEST, SALIM; 29/12/77; 18/11/25 10:12; US ABDOMEN; US; DR, REFERRER TEST; CW1082; and a checkmark in the Report column.

Break Glass Access

Users can use the Break Glass feature to access patients who have not been referred by them. Select the Break Glass icon, then search using the patient's name and date of birth.

1. In the Patient Name field, type the name in this format:

LASTNAME,FIRSTNAME (no spaces, all capitals)

2. In the Radiology UR field, enter the NHI number (all capitals)
3. Click “Break Glass”
4. Enter the patient’s Date of Birth
5. Re-enter the patient name in the same format:

LASTNAME,FIRSTNAME (no spaces, all capitals)

6. Click Search



This screenshot shows the same Voyager Imaging interface as before, but with the 'Break Glass' dialog box open in the center. The dialog box has a title bar 'Break Glass' and a close button. It contains the text 'Please enter details:' followed by two input fields: 'Enter Date of Birth (DD/MM/YYYY):' and 'Enter Patient Name (Lastname,Firstname):'. At the bottom of the dialog are 'Search' and 'Cancel' buttons. The background interface shows the search filters and the 'No matching studies.' message in the table area.

Toolbar

Hover your mouse over any toolbar button to see a description of its function.

Changing Your Password

1. Click the Settings (cog) icon in the top-right corner of the worklist.

Time: Patient Name: Radiology UR: Refresh Break Glass

VOYAGER IMAGING

VOYAGER
Logout

Change Password

Patient Name	DOB	Study Date	Study Name	Type	Referred By	UR
Test, KDI	04/03/26	04/03/26 11:08	Brain Under 6(Child)	CT		KD123456

2. Select Change Password.
3. Enter your current password.
4. Enter and confirm your new password.

Password Change

Old Password:

New Password

Confirm Password

Change Cancel

Password requirements:

- At least 8 characters.
- Must include characters from at least three of the following categories:
 - Uppercase letters (A–Z)
 - Lowercase letters (a–z)
 - Numbers (0–9)
 - Special characters
- Must not contain your username or the word 'password'.

Logging Out

1. Click Back to Worklist.
2. Select Log Out from the top-right corner of the worklist.

Time: Patient Name: Radiology UR: Refresh Break Glass

VOYAGER IMAGING

VOYAGER
Logout

Report

Patient Name	DOB	Study Date	Study Name	Type	Referred By	UR
No matching studies.						

Support

For assistance, contact Portable X-Ray
 Email: pacs@portablexray.co.nz
 Phone: 09 623 0274